



## BioCon Bladder Scanner Cleaning Instructions – Polycarbonate Casing

1. Cleaning and disinfecting outer case (housing) of the system.
  - Wipe the main body using a soft cloth dampened with a hydrogen peroxide, isopropyl alcohol or other alcohol based cleaning agent or wipes. Do not allow liquids to leak into the device while cleaning. Refer to manufacturer instructions for appropriate contact time for each cleanser.
  - Thoroughly dry the device with a clean, soft cloth before re-deployment.
  
2. Cleaning and disinfecting the surface of the probe:
  - Wipe the probe cap using a soft cloth dampened with hydrogen peroxide, isopropyl alcohol or other alcohol based cleaning agent or wipes. Refer to manufacturer instructions for appropriate contact time for each cleanser.
  - Wipe with water dampened cloth.
  - Dry with a soft, clean cloth.

### **CAUTION:**

- To avoid electric shock, disconnect the system from the AC mains and the battery.
- Do not immerse the console or the probe.
- Do not use harsh, corrosive chemicals (e.g. hydrochloric acid)
- Use bleach based cleansers sparingly if scanners are exposed to isolation patients. For repeated exposure to this patient type consider a probe sheath.



## BioCon Bladder Scanner Cleaning Instructions – Polycarbonate Casing

### **APPROVED CLEANSERS:**

- Cidex OPA, Metricide 14, Metricide 28, Metrex CaviCide, Metrex CaviWipes, Metrex CaviWipes1, CALTECH DISPATCH, PDI Sani-Cloth Plus, PDI Sani-Cloth HB, PDI Sani-Cloth AF3, PDI Sani-Cloth Bleach, PDI Super Sani-Cloth, PDI Sani-Cloth Plus, Micro-Kill Disinfectant Wipes, Micro-Kill+ Disinfectant Wipes, Micro-Kill Bleach Germicidal Wipes, Micro-Kill One Germicidal Alcohol Wipes, Micro-Kill Bleach-Free Alcohol-Free Wipes, Clorox Bleach Germicidal Wipes, Clorox Hydrogren Peroxide Wipes, Wexford Labs CleanCide Wipes, ANIOS SURFA'SAFE Premium, ANIOS WIP'ANIOS Excel, Wet Wipe Chlorine Disinfection, Wet Wipe Ethanol Disinfection, Wet Wipe Triamin Disinfection, Ecolab Actichlor Plus (NaDCC)

### **DISAPPROVED CLEANSERS:**

- Diversey Oxivir Tb Wipes, Diversey Virex Tb